Local Anaesthesia Needle

US

en: Indication for use

Intended for application of ultrasound-guided local anaesthesia prior to transvaginal follicle aspiration.

Contraindications

Do not use in the presence of, or directly after, pelvic inflammatory disease.

Product Description

The Local Anaesthesia Needle is a sterile, single use device comprised of a stainless steel needle with a unique Medisteel™ PrecisionGrip™.

The needle is equipped with a luer-lock for connecting a syringe.

Storage Instructions

Store at +2 to +30 °C.

Directions for use

The product shall be used by health care professionals within ART or gynaecology.

The patient target group is an adult or reproductive-age population that undergoes ART treatment or fertility preservation.

1. Check that the packaging is intact and the “use by” date has not expired.

2. A suitable syringe is attached to the luer-lock of the needle to help draw up the local anaesthetic into the needle. The anaesthetic is then administered in the appropriate location.

Specifications

| | |
|---|---------------------|
| Sterilized using irradiation | SAL 10 ⁴ |
| Mouse Embryo Assay, 1-cell [% embryos developed to expanded blastocyst at 96 hours] | ≥ 80 |
| Bacterial Endotoxins (LAL assay) [EU/device] | < 1.2 |
| LOT specific test results are available on the Certificate of Analysis provided with each delivery. | |

Precautions

Discard if the product or the sterile packaging is damaged. Do not insert the needle or inject local anaesthetics into the ovary or into the follicles. Do not use for intrafallopian procedures.

The Local Anaesthesia Needle is intended for single use only and SHALL NOT BE REUSED. Reuse may cause contamination, patient infection, and failed procedure.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Discard product according to standard clinical practice for medical hazardous waste and metal sharps when the procedure is finished.

Symbols glossary

** Medical devices – Symbols to be used with information to be supplied by the manufacturer (ISO 15223-1:2021)

| Symbol | Title | Reference and symbol ID |
|---------------|---------------------------------|---|
| | Consult instructions for use | **Ref no: 5.4.3 |
| | Caution | **Ref no: 5.4.4 |
| | Manufacturer | **Ref no: 5.1.1 |
| | Date of manufacture | **Ref no: 5.1.3 |
| | Use-by date | **Ref no: 5.1.4 |
| | Batch code | **Ref no: 5.1.5 |
| | Catalogue number | **Ref no: 5.1.6 |
| | Sterilized using irradiation | **Ref no: 5.2.4 |
| | Double sterile barrier system | **Ref no: 5.2.12 |
| | Do not re-use | **Ref no: 5.4.2 |
| | Temperature limit | **Ref no: 5.3.7 |
| | Medical Device | **Ref no: 5.7.7 |
| | CE Mark (Conformité Européenne) | European conformity (CE) mark with Notified Body identification number as defined in MDR 2017/745 Annex V |

Caution: Federal (US) law restricts this device to sale by or on the order of a physician or practitioner trained in its use (Rx only).

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