en: Indication for use

A general purpose liquid nitrogen container to be used with laboratory equipment or processes that require liquid nitrogen, including, for example, Rapid-i™ Kit.

Product Description

SmartBox 3™ consists of:

- a box
- an insert
- a lid

Storage Instructions

Store at +2 to +30 °C.

Directions for use

The product is intended to be used by qualified laboratory staff trained in handling liquid nitrogen.

Check that the product is intact and appropriately clean before starting the procedure. If required, wipe the inside with an appropriate disinfectant.

Always wear protective gear when working with liquid nitrogen.

Place the box on a stable work bench. Fill the box with liquid nitrogen up to 1 cm from the rim.

If used for Rapid-i Kit, place the insert on top of the box. Follow the instructions provided with Rapid-i Kit.

During the procedure, the lid can be used to reduce the evaporation of liquid nitrogen.

To dispose of unused liquid nitrogen, remove the lid and insert and pour the liquid out of the SmartBox 3 using the spout or allow it to evaporate.

Precautions

The product must not be used if it is damaged.

The product must be used according to local regulations when using liquid nitrogen.

Use safety precautions when working with liquid nitrogen, including wearing protective gear such as cryo-gloves, goggles, close-toed shoes and an apron in a well-ventilated room.

Never use solvents such as acetone, acidic or alkaline liquids etc. for cleaning.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

This product must be disposed of according to standard clinical practice.

Symbols glossary

Symbol	Title	Reference and symbol ID
	Consult instructions for use	*Ref no: 5.4.3
	Caution	*Ref no: 5.4.4 Consult Instructions for use for important cautionary information.
	Manufacturer	*Ref no: 5.1.1
	Date of manufacture	*Ref no: 5.1.3
	Batch code	*Ref no: 5.1.5
	Temperature limit	*Ref no: 5.3.7
	Medical Device	Indicates that the device is a medical device as defined in Regulation (EU) 2017/745. The device is not classified as medical device in the US.
	CE Mark (Conformité Européenne)	European conformity (CE)

* Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied (ISO 15223-1:2016).

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28349_01 SmartBox 3™ ©Vitrolife 20220617