# EmbryoGlue®

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Medium for embryo transfer.

## Contraindications

EmbryoGlue® contains recombinant human albumin (produced in yeast) and gentamicin. Do not use in patients with known hypersensitivity/allergy to any of the components.

## Product Description

EmbryoGlue is a bicarbonate buffered medium containing recombinant human albumin, hyaluronan and gentamicin as an antibacterial agent.

For use after equilibration at +37°C and 6 % CO<sub>2</sub> atmosphere.

The summary of safety and clinical performance can be found at [www.vitrolife.com](#).

## Storage instructions and stability

Store dark at +2 to +8 °C.

EmbryoGlue is stable until the expiry date shown on the container labels and the LOT-specific Certificate of Analysis.

**For Non-UE and Non-Canada: Media bottles should not be stored after opening. Discard excess media no later than two weeks after first opening.**

**EU and Canada: Media bottles can be used for up to two weeks after first opening, use aseptic technique and minimize the time outside the refrigerator. Record the opening date on the bottle. Discard excess media no later than two weeks after first opening.**

## Directions for use

The product should be used by an IVF professional.

The patient target group is an adult or reproductive-age population that undergoes fertility treatment.

**Embryo transfer:** Add approximately 1 mL of EmbryoGlue to the well of a rinsed 1-well dish. Add approximately 2 mL of EmbryoGlue to the moat of the 1-well dish. Equilibrate the dish in +37°C and 6 % CO<sub>2</sub> for 4–24 hours. Equilibrate the embryos to be transferred in the well containing EmbryoGlue for a minimum of 10 minutes in a 6% CO<sub>2</sub> environment prior to transfer. Rinse the 1 mL non-toxic syringe by drawing up and then out media from the most several times until no air bubbles are observed in the syringe. Draw up approximately 0.5 mL of the medium from the moat. Firmly attach the transfer catheter to the pre-rinsed 1 mL non-toxic syringe. Flush approximately 0.5–1.0 mL of equilibrated transfer medium from the moat through and out of the catheter. After rinsing, draw approximately 0.1 mL of EmbryoGlue from the center well and expel into the moat until approximately 20 µL is left in the syringe. Under microscopic control, gently load the embryos into the moat until approximately 5–10 µL of additional EmbryoGlue followed by a small amount of air. (The small pocket of air at the plows better imaging of the tip for ultrasound guided transfers).

For the embryo transfer, pass the tip of the catheter into the uterus approximately 1 cm from the top of the cavity and expel the embryos in a total volume of approximately 25–30 µL of medium. Slowly withdraw the catheter while maintaining steady pressure on the plunger of the syringe. Make a final microscopic examination of the catheter.

## Specifications

Aseptically filtered

Mouse Embryo Assay, 1-cell [% embryos developed to expanded blastocyst at 96 hours]

Bacterial endotoxins (LAL assay) [EU/mL]

pH tested

Osmolality tested

LOT specific test results are available on the Certificate of Analysis provided with each delivery.

## Precautions

Discard product if bottle integrity is compromised. Do not use EmbryoGlue if it appears cloudy.

To avoid contamination Vitrolife strongly recommends that media should be opened and used only with aseptic technique.

The risk of reproductive toxicity and developmental toxicity for IVF media, including Vitrolife's IVF media, have not been determined and are uncertain.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Do not for injection.

Discard the product according to standard clinical practice for medical hazardous waste when the procedures finished.

## Symbolic glossary

\* Medical devices – Symbols to be used with information to be supplied by the manufacturer (ISO 15223-1:2021).

| Symbol                                                                             | Title                                          | Reference and symbol ID                                                                                              |
|------------------------------------------------------------------------------------|------------------------------------------------|----------------------------------------------------------------------------------------------------------------------|
|  | Consult instructions for use                   | *Ref no. 5.4.3                                                                                                       |
|  | Caution                                        | *Ref no. 5.4.4                                                                                                       |
|  | Manufacturer                                   | *Ref no. 5.1.1                                                                                                       |
|  | Date of manufacturer                           | *Ref no. 5.1.3                                                                                                       |
|  | Use-by date                                    | *Ref no. 5.1.4                                                                                                       |
|  | Batch code                                     | *Ref no. 5.1.5                                                                                                       |
|  | Catalogue number                               | *Ref no. 5.1.6                                                                                                       |
|  | Sterilized using aseptic processing techniques | *Ref no. 5.2.2                                                                                                       |
|  | Single sterile barrier system                  | *Ref no. 5.2.11                                                                                                      |
|  | Temperature limit                              | *Ref no. 5.3.7                                                                                                       |
|  | Keep away from sunlight (light sources)        | *Ref no. 5.3.2                                                                                                       |
|  | Contains a medicinal substance                 | *Ref no. 5.4.2                                                                                                       |
|  | Medical Device                                 | *Ref no. 5.7.7                                                                                                       |
|  | CE Mark (Conformité Européenne)                | European conformity (CE) mark with Notified Body identification number as defined in Regulation (EU) 2017/45 Annex V |

Caution: Federal (US) law restricts this device to sale by or on the order of a physician or practitioner trained into use (Rx only).

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