EmbryoSlide+ ic8 dishes

First issue 2020.12.16, revised 2024.02.10

EN

Intended use

The EmbryoSlide+ ic8 dish is an accessory for embryo storage related to an incubator, which provides an environment with controlled temperature and gas concentrations (CO_2 and optionally O_2) for the development of gametes and/or embryos at or near body temperature.

Indications for use

The EmbryoSlide+ ic8 dish is intended for culturing, handling and preparation for transfer of human gametes and/or embryos. The EmbryoSlide+ ic8 dish must be used together with the EmbryoScope+ or EmbryoScope 8 incubator.

Contraindications

None known.

Intended users

Embryologists, other laboratory personnel and clinic staff at IVF clinics trained by Vitrolife A/S-certified instructors.

Intended patient group

Patients undergoing infertility treatment.

Clinical benefits

- Improved embryo development
- Improved implantation/pregnancy rate
- Reduced pregnancy loss rate.

Limitations and restrictions on use

The EmbryoSlide+ ic8 dish is only intended for use in the EmbryoScope+ and EmbryoScope 8 incubators and must be handled by trained personnel according to the instructions in this insert. Any serious incident that has occurred in relation to the dish should be reported to Vitrolife and to the competent authority of the Member State in which the user is established.

The EmbryoSlide+ ic8 dishes and lids are sterile. The pouches must only be opened in a sterile laminar flow hood, and the dish must be covered by the lid when in use.

Warnings and precautions

WARNINGS

The EmbryoSlide+ ic8 dish is intended for single use only and MAY NOT be re-used. Any attempt by the user to clean and re-sterilise the dish may result in contamination with microorganisms or other risks of device failure.

To avoid contamination with microorganisms, always place the dish in a sterile laminar flow hood while loading and generally handling the dish.

Always label the dish appropriately and validate the label when the embryo is transferred either to a new device or to the patient.

PRECAUTIONS

Do not use the rinsing wells for incubating embryos as no images are acquired from these wells.

The reservoir and rinsing wells MUST always be covered by a confluent oil layer of 1.6 ml of IVF-grade oil. This prevents the medium from evaporating during incubation.

Evaporation of the medium can change osmolality, which may affect embryo development.

Always place the dish in a sterile laminar flow hood while loading it. Take care while loading and handling the dish to reduce the risk of spilling any oil or medium.

If ANY oil or medium is accidentally spilled from the dish while it is loaded or handled, the embryos MUST be transferred to a new dish to prevent them from being adversely affected by the spillage.

In case any oil is spilled on the dish outside of the reservoir, gently wipe off the oil by using a lint-free tissue paper. If any oil is present on the lid, the lid must be replaced.

If any bubbles are present in the well or oil layer after preparation, they may block the camera light and compromise image quality. In rare cases, bubbles may dislodge the embryo from the bottom of the microwell.

Any bubbles must be removed carefully and relatively quickly to avoid evaporation of medium.

Take care not to remove any medium when removing the bubbles.

Discard the product according to standard clinical practice for medical hazardous waste when the procedure is finished.

Product description

The EmbryoSlide+ ic8 dish is a single-use sterile dish designed for the culture of embryos in the EmbryoScope+ and EmbryoScope 8 incubators.

The EmbryoSlide+ ic8 dish contains two types of wells: eight culture wells in which the embryos reside during incubation and four rinsing wells that are used for rinsing and handling the embryos. The embryos reside in a central depression – the microwell – inside each culture well. The culture wells are cylindric and have a diameter of 3.45 mm. The microwells are conic with a curved bottom and have a bottom diameter of 0.28 mm. See the illustration on the last page of this insert for an overview of the dish.

Preparing the dish

Prepare one dish at a time to minimise handling time. Work on a non-heated workbench and use cold medium and oil.

1. Remove the dish from the pouch in a sterile laminar flow hood.
2. Fill all microwells with medium using a micropipette (max. diameter: 200 μm). One filling of the tip will suffice to

completely fill all microwells. The tip of the pipette must touch the side of the microwell during filling. Slightly overfill each microwell.

3. Use a standard pipette to immediately fill an additional 20 μl of medium into each well (160 μl in total to fill all eight wells).
4. Fill each rinsing well with max. 30 μl and min. 25 μl of medium.
5. Load 1.6 ml of oil certified for IVF into the reservoir. Make sure that all culture wells and rinsing wells are covered with a confluent oil layer. Add an additional amount of oil for each well or reservoir not filled with medium.
6. Push up any large bubbles with a micropipette and then remove them.
7. Cover the dish with the lid and equilibrate it overnight.
8. Identify and remove any bubbles under a stereo microscope.
9. Place the barcode label on the dedicated label tab on the dish. The barcode must be placed so that it is possible to read the text on the label when the dish is correctly inserted into the incubator. Make sure that the label is smoothed out as best as possible to avoid wrinkles in the paper. If the barcode is in any way damaged or not smoothly applied, the label cannot be read. In this case, print a new label from the EmbryoViewer software.
10. Load embryos into the centre of the microwells using a micropipette.
11. Place the dish in the incubator.

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