G-TL™

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US

en: Indication for use

Medium for culture of embryos from fertilization to the blastocyst stage.

Contraindications

G-TL™ contains gentamicin. Do not use in patients with known hypersensitivity/allergy to the component.

Product Description

SUPPLEMENTED WITH HSA

G-TL is a bicarbonate buffered medium containing human serum albumin, hyaluronan and gentamicin as an antibacterial agent.

Ready to use after equilibration at +37 °C and 6 % CO₂ atmosphere.

The summary of safety and clinical performance can be found at [www.vitrolife.com](#)

Storage instructions and stability

Store dark at +2 to +8 °C.

G-TL is stable until the expiry date shown on the container labels and the LOT-specific Certificate of Analysis.

Media bottles can be used for up to two weeks after first opening, use aseptic technique and minimize the time outside the refrigerator. Record the opening date on the bottle. Discard excess media no later than two weeks after first opening.

US: Media bottles should not be stored after opening. Discard excess media after completion of the procedure.

Directions for use

The product shall be used by an IVF professional.

The patient target group is an adult or reproductive-age population that undergoes IVF treatment or fertility preservation.

To use G-TL with time-lapse devices follow the manufacturer's instructions for use.

The following is the general procedure for embryo culture using G-TL.

Use a pre-rinsed sterile tip and place an appropriate number of droplets of G-TL into the dish. Immediately cover the drops with OVOIL™ to avoid evaporation. Place the dish in the incubator at 6 % CO₂ and +37 °C. Dishes should be equilibrated for no less than 6 and no more than 18 h before use.

Following fertilization, wash the embryos before placing into G-TL. To avoid depletion of nutrients a minimum of approximately 10µl of G-TL per embryo should be used. Return the dish to the incubator immediately. Embryos can be transferred to the uterus in EmbryoGluo® or in G-TL (not cleared for embryo transfer in the US).

Specifications

Asepticly filtered

Mouse Embryo Assay, 1-cell [% embryos developed to expanded blastocyst at 96 hours]

Bacterial endotoxins (LAL assay) [EU/mL]

pH tested

Osmostically tested

LOT specific test results are available on the Certificate of Analysis provided with each delivery.

Precautions

Discard product if bottle integrity is compromised. Do not use G-TL if it appears cloudy.

G-TL contains human serum albumin.

Caution: All blood products should be treated as potentially infectious. Source material from which this product was derived was found negative when tested for antibodies to HIV, HBs, HCV, and HTLV III and non-reactive for HbsAg, HCV RNA and HIV-1 RNA and syphilis. No known test methods can offer assurance that products derived from human blood will not transmit infectious agents.

To avoid contamination Vitrolife strongly recommends that media should be opened and used only with aseptic technique.

The risk of reproductive toxicity and developmental toxicity for IVF media, including Vitrolife's iVF media, have not been determined and are uncertain.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. Do not injection.

Discard the product according to standard clinical practice for medicinal hazardous waste when the procedure is finished.

Symbols glossary

* Medical devices – Symbols to be used with information to be supplied by the manufacturer (ISO 15223-1:2021).

Symbol	Title	Reference and symbol ID
	Consult instructions for use	*Ref no: 5.4.3
	Caution	*Ref no: 5.4.4
	Manufacturer	*Ref no: 5.1.1
	Date of manufacture	*Ref no: 5.1.3
	Use-by date	*Ref no: 5.1.4
	Batch code	*Ref no: 5.1.5
	Catalogue number	*Ref no: 5.1.6
	Sterilized using aseptic processing techniques	*Ref no: 5.2.2
	Single sterile barrier system	*Ref no: 5.2.11
	Temperature limit	*Ref no: 5.3.7
	Keep away from sunlight (light sources)	*Ref no: 5.3.2
	Contains a medicinal substance	*Ref no: 5.4.7
	Contains human blood or plasma derivatives	*Ref no: 5.4.7
	Medical Device	*Ref no: 5.7.7
	CE Mark (Conformité Européenne)	European conformity (CE) mark with Notified Body identification number as defined in Regulation (EU) 2017/745 Annex V

Caution: Federal (US) law restricts this device to sale by or on the order of a physician or practitioner trained in its use (Rx only).

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