## US

**EN: Instructions for use – MicroCell™ Disposable Counting Chamber****Intended Us**

MicroCell™ Counting Chambers are non-sterile, disposable, glass microscope slides designed for semen concentration and motility analysis in vitro.

Reference	Product Description
15423	MicroCell™ 20µ chamber depth, 2-chamber, 24 slides per box
15424	MicroCell™ 20µ chamber depth, 4-chamber, 24 slides per box

**Directions for use**

This product must be used under the guidance of personnel who are well-qualified in andrology procedures. For single use only.

If packaging is damaged, do not use.

Cut the seal on two sides of the box. Lift the top lid off the box to expose the glass slides.

**Neat Sample Preparation:**

Allow the semen sample to completely liquefy (approx. 20 minutes at room temperature) and thoroughly mix the sample according to established methods used at the laboratory.

**Post Washed Samples:**

MicroCell™ is intended for use with unwashed semen. For use with a washed semen sample, add 0.3% HSA to avoid adherence of sperm to the glass slide.

**Loading the MicroCell™:**

- Using a positive displacement pipette, place approx. 2.0-2.5 µl of the sample to be evaluated into one of the clear loading zones:

Approximate full volume by slide:

a. 15423, MC-20-2 = 3 µl

b. 15424, MC-20-4 = 2 µl

- Allow a few seconds for the sample to load into the analysis area, and then remove any excess fluid remaining in the loading zone.

- Perform the analysis near the center of the large open area of the MicroCell™.

Manual analysis is performed in conjunction with a microscope and an eyepiece reticle containing the counting grid, which is used to analyze the patient sample via microscopy. The counts are then used to calculate concentration and motility of the patient sample. Computer assisted analysis can also be obtained using the MicroCell™. For more information on the recommended use of MicroCell™ Counting Chambers, please see "MicroCell™ User Manual". The Manual is available on request from Vitrolife, and through www.vitrolife.com.

**Precautions**

Discard the product if the product or packaging is damaged.

MicroCell™ Disposable Counting Chambers are intended for single use only and may not be reused. Reuse may cause contamination, and failed procedure.

Do not use broken slides.

Once the MicroCell™ has been loaded with patient sample the slide should be considered biohazardous material, therefore, universal precautions should be used to avoid occupational exposure.

**Disposal Instructions**

The MicroCell™ loaded with patient sample should be considered clinical or medical waste. Proper disposal should be according to national or regional regulations per regulated medical waste procedures.

**Specifications**

Chamber depth confirmed by laser interferometry

Nonsterile

Store at room temperature

**Symbols**

	In Vitro Diagnostic
	Do not re-use, discard after procedure.
	Use by
	Caution: Consult accompanying documents.
	Catalog number

	Batch code
	Date of Manufacture

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Vitrolife Sweden AB, Box 9090 (Gustaf Werner gata 2), SE-400 92 Göteborg, Sweden.
Vitrolife Inc., 3601 South Inca Street, Englewood CO 80110, USA.
Vitrolife Inc.,