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RUSSIA

Mouse Embryo Assay, 1-cell [% embryos developed to expand blastocysts at 96 hours]	≥80
Micro droplet Mouse Embryo Assay, 1-cell [% embryos developed to expanded blastocyst at 96 hours]	PASS
Human Sperm Oil Assay	PASS
Bacterial endotoxins (LAL assay) [EU/mL]	< 0.25
LOT specific test results are available on the Certificate of Analysis provided with each delivery.	

Discard product if bottle integrity is compromised. Do not use OVOIL if it appears cloudy.	
Re-use may result in microbiological contamination and/or property changes in the product.	
To avoid contamination Vitrolife strongly recommends that media should be opened and used only with aseptic technique.	
The risks of reproductive toxicity and developmental toxicity for iVF media, including Vitrolife's iVF media, have not been determined and are uncertain.	
Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.	
Discard product according to standard clinical practice for medical hazardous waste when the procedure is finished.	

Discard product if bottle integrity is compromised. Do not use OVOIL if it appears cloudy.	
Re-use may result in microbiological contamination and/or property changes in the product.	
To avoid contamination Vitrolife strongly recommends that media should be opened and used only with aseptic technique.	
The risks of reproductive toxicity and developmental toxicity for iVF media, including Vitrolife's iVF media, have not been determined and are uncertain.	
Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.	
Discard product according to standard clinical practice for medical hazardous waste when the procedure is finished.	

Discard product if bottle integrity is compromised. Do not use OVOIL if it appears cloudy.	
Re-use may result in microbiological contamination and/or property changes in the product.	
To avoid contamination Vitrolife strongly recommends that media should be opened and used only with aseptic technique.	
The risks of reproductive toxicity and developmental toxicity for iVF media, including Vitrolife's iVF media, have not been determined and are uncertain.	
Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.	
Discard product according to standard clinical practice for medical hazardous waste when the procedure is finished.	

* Medical devices – Symbols to be used with information to be supplied by the manufacturer (ISO 15223-1:2021).

Symbol	Title	Reference and symbol ID
	Manufacturer	*Ref no: 5.1.1
	Date of manufacture	*Ref no: 5.1.3
	Use-by date	*Ref no: 5.1.4
	Batch code	*Ref no: 5.1.5
	Catalogue number	*Ref no: 5.1.6
	Sterilized using aseptic processing techniques	*Ref no: 5.2.2
	Single sterile barrier system	*Ref no: 5.2.11
	Keep away from sunlight (light sources)	*Ref no: 5.3.2
	Temperature limit	*Ref no: 5.3.7
	Do not re-use	*Ref no: 5.4.2
	Consult instructions for use	*Ref no: 5.4.3
	Caution	*Ref no: 5.4.4
	Medical Device	*Ref no: 5.7.7
	CE Mark (Conformité Européenne)	

Caution: Federal (US) law restricts this device to sale by or on the order of a physician or practitioner trained in its use (Rx only).

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EUROPE

en: Indication for use

Oil for covering of medium during in vitro fertilisation and micro-manipulation procedures.

Contraindications

None known.

Product Description

OVOIL™ is a sterile, light paraffin oil.

For use after equilibration at +37°C and 5 % CO₂, 6 % CO₂ or ambient atmosphere, depending on the intended use.

The summary of safety and clinical performance can be found at [www.vitrolife.com](#)

Storage instructions and stability

Store dark at +20 ± 8 °C.

OVOIL is stable until the expiry date shown on the container labels and the LOT-specific Certificate of

Analysis.

Media bottles should not be stored after opening. Discard excess media after completion of the procedure.

Directions for use

The product shall be used by an iVF professional.

The patient target group is an adult or reproductive-age population that undergoes fertility treatment.

Remove OVOIL from vial using a sterile, non-toxic pipette. Do not pour.

Using a sterile, non-toxic pipette, add culture medium to a vessel labelled with patient ID. For large volume culture, add 0.5 to 1.0 mL to a sterile tube or well and cover entirely with OVOIL. For small volume culture, place 10 to 100 µL droplets in a sterile culture dish and cover entirely with OVOIL.

Rinse the catheter in case of aspiration of embryo(s) from medium under oil.

Specifications

Asepticly filtered

Mouse Embryo Assay, 1-cell [% embryos developed to expand blastocysts at 96 hours]

Micro droplet Mouse Embryo Assay, 1-cell [% embryos developed to expanded blastocyst at 96 hours]

Human Sperm Oil Assay

Bacterial endotoxins (LAL assay)

LOT specific test results are available on the Certificate of Analysis provided with each delivery.

Precautions

Discard product if bottle integrity is compromised. Do not use OVOIL if it appears cloudy.

Re-use may result in microbiological contamination and/or property changes in the product.

To avoid contamination Vitrolife strongly recommends that media should be opened and used only with aseptic technique.

The risks of reproductive toxicity and developmental toxicity for iVF media, including Vitrolife's iVF media, have not been determined and are uncertain.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Discard product according to standard clinical practice for medical hazardous waste when the procedure is finished.

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