Ultra RapidWTM Blast

Vitrolife

EUROPE

en: Indication for use

Medium for warming of vitrified human blastocysts and embryos.

Contraindications

Ultra RapidWTM Blast contains gentamicin. The product can only be used in contact with embryos. Ensure appropriate precautions are taken to minimize the risk of contact with individuals with known hypersensitivity to gentamicin.

Product Description

Ultra RapidWTM Blast contains gentamicin. The product can only be used in contact with embryos. Ensure appropriate precautions are taken to minimize the risk of contact with individuals with known hypersensitivity to gentamicin.

Storage instructions and stability

Store dark at 2 °C - 8 °C.

Ultra RapidWTM Blast is stable until the expiry date shown on the bottle label and the LOT specific Certificate of Analysis.

Media bottles can be used for up to two weeks after first opening, using aseptic technique and minimize the time between opening and use.

Do not use Ultra RapidWTM Blast in a CO₂ environment.

The following is the general procedure for using Ultra RapidWTM Blast.

Warning: Warming procedures are performed at 37 °C.

1. Place at least 0.5 ml of warming solution into a dish and warm to 37 °C for a minimum of 15 minutes or a heated water bath.

The time may vary and must be validated by the laboratory.

2. Prepare the vitrification device containing the vitrified blastocysts according to its Instructions for Use.

3. Remove the vitrification device from the liquid nitrogen and immediately plug the tip into the warming solution.

4. Leave the vitrified(?) for 2 minutes.

5. Wash the blastocyst(s) several times with culture medium, then incubate until further use.

Specifications

Sterile No evidence of microbial growth.

Mice Embryo Assay, one-cell system [% embryos developed to expanded blastocyst at 96 hours] ≥ 80

Bacterial endotoxins (LAL assay) [EU/ml] < 0,5

pH tested

Osmolality tested

LOT specific test results are available on the Certificate of Analysis provided with each delivery.

Precautions

Ultra RapidWTM Blast contains human serum albumin and gentamicin.

Caution: All blood products should be treated as potentially infectious. Source material from which this product was derived has been found negative when tested for antibodies to HIV, HBV, HCV and HTLV-I and -II. No known test methods can offer assurance that products derived from human blood will not transmit infectious agents.

Discard product if bottle appears cloudy. Do not resterilize after opening.

Discard the product according to standard clinical practice for medical waste when the procedure is finished.

Currently research literature indicates the long-term effects of warming vitrified embryos remain unknown.

The risk of reproductive toxicity and developmental toxicity for IVF media, including Vitrolife's IVF media, have not been determined and are uncertain.

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